


CLICKIT ER PELVIS

PELVIC FIXATOR

Universal clamp

Rapid application

Sterile kit

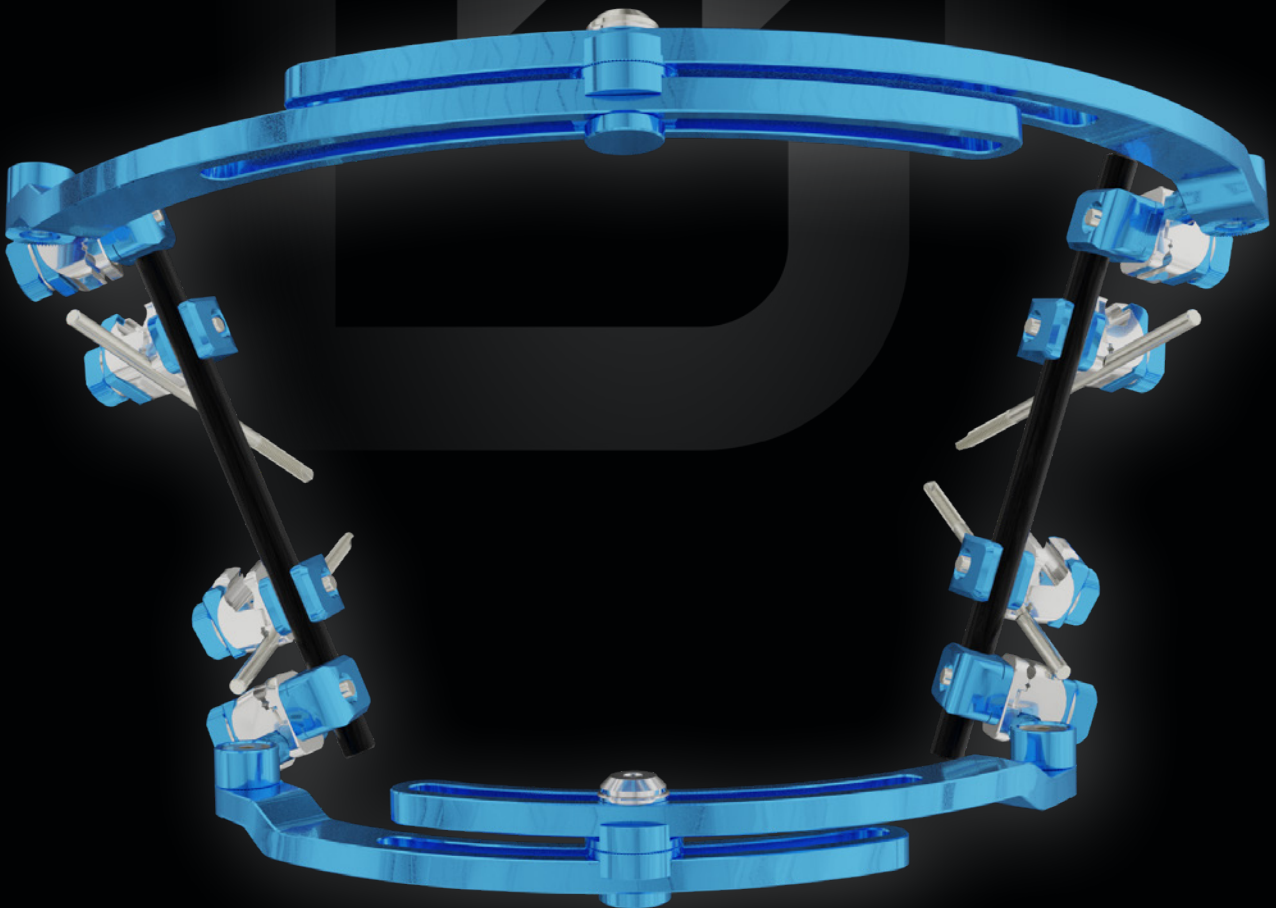

Indications

Emergency stabilization of complete pelvic instability.

FEATURES

CLINICAL ADVANTAGES

Universal clamp

Multiplanar stabilization

Snap and click coupling

Sterile

rod/rod, rod screw (Ø 4.0-5.0-6.0 mm) and rod wire coupling

Increased stability

Quick application

Ready for use

Components



Universal clamp


One clamp for all types of coupling



Rod/Rod Coupling



Rod/Screw coupling

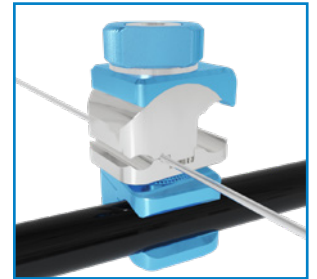

Rod/wire coupling



12mm Rods



Pelvic arches

Made of aluminum alloy.
Available in two sizes: large for the proximal side and small for the distal side



Self-drilling Cortical screws

Kit ClickIt ER Pelvis



Description	Q.ty
Large arch (proximal)	1
Small arch (distal)	1
Universal clamps	8
Carbon rods Ø 12 mm x 250 mm	2
Self-drilling cortical screws Ø 6.0 mm L 250 mm thread 55 mm	4
Dedicated instrumentation	

Configuration



Sterilization

The components of the ClickIt ER system are supplied sterile.

We indicate sterilizing the instruments in saturated steam autoclave according to the following parameters:

TEMPERATURE: 134°

EXPOSURE: 10 min

CLICKIT ER PELVIS 5006542ST

COMPONENTS AND ACCESSORIES

Code	Description
5006508ST	Universal clamp
5006504ST	ER multiscrew clamp
5006510ST	Rod Ø 12 L 100 mm
5006512ST	Rod Ø 12 L 150 mm
5006514ST	Rod Ø 12 L 200 mm
5006516ST	Rod Ø 12 L 250 mm
5006518ST	Rod Ø 12 L 300 mm
5006520ST	Rod Ø 12 L 350 mm
5006522ST	Rod Ø 12 L 400 mm
5006526ST	Rod Ø 12 L 500 mm
5006528ST	Rod Ø 12 L 600 mm

INSTRUMENTS *

Code	Description	Q.ty
5006562	Complete ClickIt ER instrument trax	
Composition:		
5000124	3.0 mm hex screwdriver	1
5000222	Twist drill Ø 3.5 mm L 200 mm	1
5000600	T-wrench for screws Ø 4.0 mm	1
5000601	T-wrench for screws Ø 5.0 mm	1
5000602	T-wrench for screws Ø 6.0 mm	1
5000604	T-handle with AO attachment	1
5000605	Hexagonal rod Ø 5.0 mm AO connection	1
5006.04	Ø 4.0 mm adapter with AO connection	1
5006.05	Ø 5.0 mm adapter with AO connection	1
5006.06	Ø 6.0 mm adapter with AO connection	1
5000622	5.0 mm L-shaped Allen key	1
5006000	3.0 mm L-shaped Allen key	1
5006010	Drill guide Ø 3.5 mm L 110 mm	1
5006564	ER rod template	1
5006560ER	ClickIt ER sterilization trax	1

* Implantable and sterile devices are covered by the CE-1936 marking, while non-sterile reusable instruments are covered by the CE marking



HEADQUARTERS

Via P. Gobetti, 56r - 16145 Genova
Tel. +39 010 30801 - Fax +39 010 3080210
www.mikai.it - servizio.clienti@mikai.it

MANUFACTURING FACILITY

Via Canestrello, 2 - 36050 Monteviale (VI)
Tel. +39 0444 950100 - Fax +39 0444 950133
www.mikai.it - servizio.clienti@mikai.it