

VCA

CONICAL TALUS SCREW


Stainless steel and titanium
Cannulated
Sterile




Surgical technique




With the patient in a supine position and an inversion of the foot, perform about a 1 cm incision near the sinus tarsi.

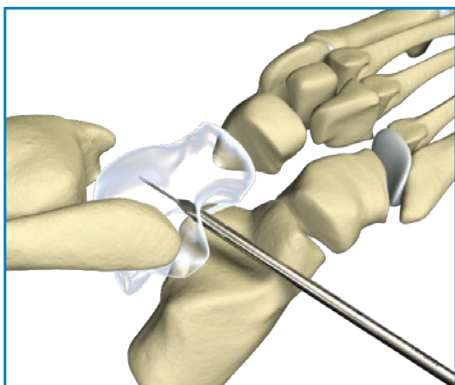

Under x-ray while maintaining a manual correction of the talocalcaneonavicular joint, insert a guide wire, closely behind the calcaneal region with the direction aimed at the center of the tibiotalar joint (about 60° with respect to the support with either the sagittal plane or the frontal plane). Perform a preliminary perforation of the talus with a punch just inside the tuberositas facet to prepare an access in the cortical bone for the screw insertion.

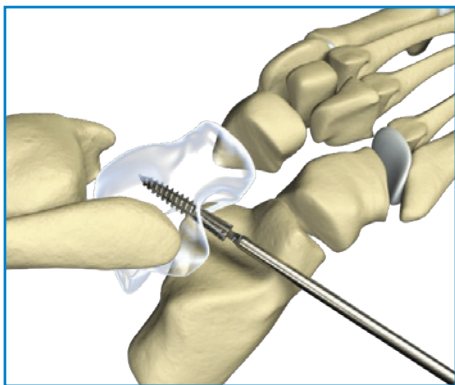

A screw is selected based on the foot size, with lengths from 25 to 45 mm, and proceed with its introduction. Adjust the depth of the screw to the height of the protrusion suitable to assure the correction and suture the skin.

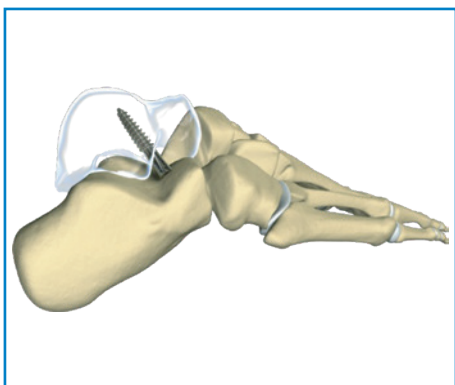

The procedure consistently yields favorable outcomes and requires minor post-operative effort from the patient.

Codes

VCA - Stainless steel

Code	Description
5000411ST	Talar conical screw Ø 6.5 mm L 25 mm
5000412ST	Talar conical screw Ø 6.5 mm L 30 mm
5000413ST	Talar conical screw Ø 6.5 mm L 35 mm
5000401ST	Talar conical screw Ø 8.0 mm L 30 mm
5000402ST	Talar conical screw Ø 8.0 mm L 35 mm
5000403ST	Talar conical screw Ø 8.0 mm L 40 mm
5000404ST	Talar conical screw Ø 8.0 mm L 45 mm
5000405ST	Talar conical screw Ø 8.0 mm L 50 mm

VCA - Cannulated stainless steel

Code	Description
5000411FST	Talar conical cannulated screw Ø 6.5 mm L 25 mm
5000412FST	Talar conical cannulated screw Ø 6.5 mm L 30 mm
5000413FST	Talar conical cannulated screw Ø 6.5 mm L 35 mm
5000401FST	Talar conical cannulated screw Ø 8.0 mm L 30 mm
5000402FST	Talar conical cannulated screw Ø 8.0 mm L 35 mm
5000403FST	Talar conical cannulated screw Ø 8.0 mm L 40 mm
5000404FST	Talar conical cannulated screw Ø 8.0 mm L 45 mm
5000405FST	Talar conical cannulated screw Ø 8.0 mm L 50 mm

VCA - Titanium

Code	Description
5000411TST	Talar conical titanium screw Ø 6.5 mm L 25 mm
5000412TST	Talar conical titanium screw Ø 6.5 mm L 30 mm
5000413TST	Talar conical titanium screw Ø 6.5 mm L 35 mm
5000401TST	Talar conical titanium screw Ø 8.0 mm L 30 mm
5000402TST	Talar conical titanium screw Ø 8.0 mm L 35 mm
5000403TST	Talar conical titanium screw Ø 8.0 mm L 40 mm
5000404TST	Talar conical titanium screw Ø 8.0 mm L 45 mm
5000405TST	Talar conical titanium screw Ø 8.0 mm L 50 mm

INSTRUMENTS

Code	Description
5000430	Hexagonal screwdriver 3.5 mm cannulated
5000431	Cannulated twist drill VCA
5000410	Single guide wire Ø 1.5 mm x 200 mm VCA

**HEADQUARTERS**

Via P. Gobetti, 56r - 16145 Genova
Tel. +39 010 30801 - Fax +39 010 3080210
www.mikai.it - servizio.clienti@mikai.it

MANUFACTURING FACILITY

Via Canestrello, 2 - 36050 Monteviale (VI)
Tel. +39 0444 950100 - Fax +39 0444 950133
www.mikai.it - servizio.clienti@mikai.it